

## PROTECHTM EMERY POWDER SUPERIOR POLISHING OF ACRYLICS AND RESINS

## **ADVANTAGES OVER PUMICE:**

- <u>Superior Results</u> Greater satisfaction with end product by technician, doctor, and patient
- <u>Faster Polishing</u> Less time spent at the polishing wheel results in less labor cost, producing greater profit margin
- Longer Lasting and Cost Effective Lasting longer then pumice, the product costs no more then current materials
- Odorless No longer need to contend with the foul smell of pumice
- Consistent Same excellent results every time

Requires no additional learning curve since the procedure and equipment are the same that is used with pumice

ProTech™ Emery Powder is all natural and friendly to the environment. Use for the polishing of dentures, partials, provisional bridges and any other resin prosthesis.

## **INSTRUCTIONS:**

Mix 8 ½ oz. water with 1 pound of Emery Powder to obtain optimum results.

## **WARNING:**

This product contains crystalline silica. When pouring powder, wear a face mask. Prolonged breathing of excessive concentrations of dust from this product may cause lung damage. Do not inhale the dust. In case of pollution, use a respirator such as a 3M TM, 9900 or MSA TM Ultra-twin with H filter.

Avoid getting powder in the eyes. If eye contact occurs, rinse thoroughly with cold water. If irritation persists consult a physician.